




BARRIER® Lithotomy set

- Effective absorption helping to maintain a dry working area
- Guided handling points and sophisticated folding principles ensure aseptic application in the draping procedure
- 100% impermeable materials providing optimal patient safety
- Easy for one person to drape
- User-friendly packaging with handles on the dispenser case for efficient handling.

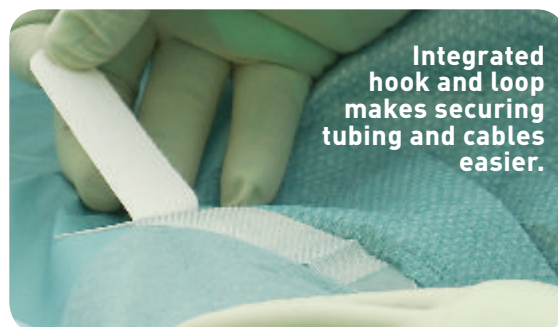

Draping instruction



1. Put your hands inside the folding and open the under buttock drape fully. Place it under the patient.



2. Put your hands into the fold of the lithotomy legging. Unfold all the way over the leg. If necessary fold the exposed film part into the legging. Do the same for the other leg.



3. Note the direction of the pictogram on the lithotomy drape. Remove the release paper. Place the lithotomy drape in the centre of the genital area. Unfold towards the head and towards yourself. Ensure that the adhesive film is securely attached to the patient and remove the crepe paper.



4. Draping is then completed. Use the hook and loop to secure tubing and cables.

Scan the QR code for more about Mölnlycke Health Care or to watch a draping instruction video.



Ref. No.	Description	Disp/Trp
61010	Lithotomy set 1 under buttocks drape 75x100 cm 2 lithotomy leggings 75x120 cm 1 lithotomy drape 75/150x170 cm, aperture 9x11 cm 1 table cover 150x190 cm, 75x190 cm absorbent	6/12
61020	Lithotomy set 1 under buttocks drape 75x100 cm 2 lithotomy leggings 75x120 cm 1 lithotomy drape 75/150x170 cm, aperture 9x11 cm 1 wrap 90x90 cm	9/18
61000	Lithotomy drape 75/150x170 cm, aperture 9x11 cm	20/40

All specified set contents are also available for inclusion in ProcedurePak® trays.

Ref. No.	ProcedurePak components
71000	Lithotomy drape 75/150x170 cm, aperture 9x11 cm

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, SE-402 52 Göteborg, Sweden.
T +46 31 722 30 00. F +46 31 722 34 00. www.molnlycke.com

The Mölnlycke Health Care, BARRIER®, and ProcedurePak® names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Copyright (2012)