


Uro/gynaecology Set

- 100 % impermeable all over
- Fast all-in-one draping, easy for one person to drape
- Fluid control through integrated collection pouch
- Self-adhesive edges around the genitalia opening
- Integrated folds near the operative site to facilitate the fixation of sutures
- Integrated fixation of the leggings to the sides

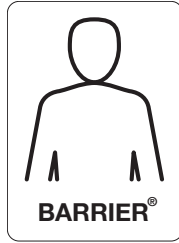




01 First place the under buttock drape under the patient. Then remove the release paper from each side and the lower part of the a-i-o drape.

- The label shows the direction of the drape.

02 Start to fix the lower part and the side of the a-i-o drape. Remove the upper part of the release paper.

- The label shows the unfolding towards the feet.

03 Hold the left hand on the drape and unfold the right legging over the patient's leg (see label).

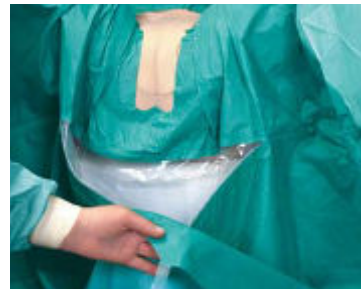

04 Unfold the left legging over the patient's leg.

05 Loosen the tape and unfold the lower part of the drape - can also cover a table if needed.

06 Unfold the upper part of the drape over the patient and anaesthetic frame. The draping is finished.

07 The integrated 'folds' can be used for sutures that have been fixed to the patient or for the fixation of instruments.

Uro/gynaecology Set / Ref. No. 905020	Size (cm)
1 mayo stand cover	79 x 145
1 under buttocks drape	75 x 100
4 cellulose towel	19 x 25
1 uro/gynaecology drape	200/355 x 295
1 table cover	150 x 190

Pack Count: 5/10

