

Beach Chair Shoulder Drape

- Impermeable all over to minimise risk of strikethrough in all surgical situations
- Generously sized all-in-one design for shoulder surgery performed in beach chair/sitting position
- Integrated fluid collection pouch for effective fluid control for right or left shoulder procedures
- Integrated hooks and loops for optimal cord management





1 Place an approach split sheet.



2 Place the adhesive split sheet at the broadest part of the operation area.



3 Remove the release paper from adhesive edges.



4 Unfold the drape to the first fold. Position the drape towards the patient as indicated by exit port directional arrow.



5 Place the patient's arm through the elastic aperture.



6 Fix the adhesive to the patient's shoulder and unfold the drape one side at a time.



7 After locating the leading edges on the inside of the top first fold, unfold the drape toward the patient's feet and then over the patient's head towards the anesthetic frame.



8 Adjust fluid control pouch as desired. The drainage port may be connected to suction tubing if needed.

Sterile codes	
60004	Beach Chair Shoulder Drape 400×248 cm, ap.10×10 cm, elastic, wrapped
60003	Beach Chair Shoulder Set
996109	1 mayo stand cover reinforced 79×145 cm
-	4 cellulose towel 19×25 cm
9475	1 split sheet (plastic) 150×193 cm, split 15×61 cm
946111	1 stockinet 22×75 cm
911500	2 op-tape 9×49 cm
70004	1 beach chair shoulder drape 400×248 cm, aperture 10×10 cm elastic
948090	1 table cover reinforced 150×190 cm
ProcedurePak® Component	
70004	BNS Beach Chair Shoulder Drape 400×248 cm, ap.10×10 cm elastic