

## Extremity Drape, bilateral foot, w/ Drisite® Plus

- New extremity drape with Drisite Plus for outstanding fluid control in wet surgery
- Impermeable materials all over to protect from strikethrough in all surgical situations
- Reinforced materials in the critical zone minimise tearing and risk of breach in the infection control system
- The new BARRIER® surgical drapes feature soft and comfortable materials that conform naturally to the body





1 Unfold the table cover and place it under the patient's legs.



2 Place the unfolded extremity drape over the patient's feet with the instruction label pointing up towards the patient's body.



3 Reach through the elastic opening and grasp the patient's feet. Unfold the drape over the foot end of the op-table and then upwards.



4 Unfold the upper part of the drape over the anesthetic frame.

Sterile code	
60300	Extremity set, bilateral foot
-	4 cellulose towel 19×25 cm
911510	2 op-tape 9×49 cm
70311	1 bilateral foot drape 200×300 cm, apertures 3.5 cm elastic
948090	1 table cover reinforced 150×190 cm
ProcedurePak® Components	
70311	BNS Bilateral foot drape 200×300 cm, apertures 3.5 cm elastic, patched

