

Extremity Drape w/ Drisite® Plus

- The new BARRIER® extremity drapes are purpose-designed featuring only impermeable materials to minimise the risk of strikethrough in all surgical situations
- Ultra strong material performance in critical area
- The patented Drisite Plus system enhances drape absorption to minimise risk of bacteria transfer and unwanted liquids on the OR floor
- The new BARRIER surgical drapes feature soft and comfortable materials that conform naturally to the body





1 Unfold the table cover and place it under the patient's leg.



2 Alt. 1: Cover the lower leg with the adhesive towel and fix it with op-tape.



3 Alt. 2: Cover the leg with the small stockinet.



4 Fix it with op-tape.



5 Position the unfolded extremity drape over the patient's foot with the arrow pointing up towards the patient's body.



6 Unfold one side of the extremity drape and hand it to an assistant. Lift the patient's leg and pull the extremity drape up over the patient's knee. Unfold the sides of the drape.



7 Reach through the elastic opening, grasp the patient's foot and pull the drape down over the foot end.



8 Unfold the upper part of the drape over the anesthetic frame.

Sterile codes	
60200	Extremity Set
60202	Extremity Set
60203	Extremity Set
60204	Extremity Set
60205	Extremity Set, wrapped Extremity Set
60207	Extremity Set
60208	Extremity Set
60209	Extremity Set
60211	Extremity Drape 200×300 cm, ap.7 cm elastic, patched
60212	Extremity Drape 230×315 cm, ap.7 cm elastic, patched, wrapped
60213	Extremity Drape 230×315 cm, ap.7 cm elastic, patched
ProcedurePak® Components	
70210	BNS Extremity Drape 250×315 cm, ap.7 cm elastic, patched
70211	BNS Extremity Drape 200×300 cm, ap.7 cm elastic, patched
70213	BNS Extremity Drape 230×315 cm, ap.7 cm elastic, patched

