

Extremity Set

- Extra resistant material with strong reinforcement at the operative site
- Improved fluid absorbency around the incision area
- Integrated tube holders with improved resistance
- Completely impermeable material
- Accurate folding and packing order to allow aseptic application of the product
- Absorbent comfort layer for the patient





01 Unfold the table cover and place it under the patient's leg.



02 Cover the lower leg with the adhesive towel and fix it with op-tape.



03 The leg can also be covered with a stockinet (small)



04 Place the unfolded extremity drape over the patient's foot with the arrow pointing up towards the patient's body.



05 Reach through the elastic opening, grasp the patient's foot and pull the drape down over the foot.



06 Alternative method: When two scrubbed assistants are available the extremity drape is first pulled up over the patient's knee, then sides are unfolded followed by the lower end and finally the upper end.



07 Unfold one side of the extremity drape and hand it to an assistant. Lift the patient's leg and pull the extremity drape up over the patient's knee. Unfold the sides of the drape.



08 Then unfold the lower end under the patient's leg. Unfold the upper part of the drape over the anaesthetic frame. Attach the light cord and water supply unit into the tubeholders.

Extremity Set / Ref. No. 697400	Size (cm)
1 mayo stand cover reinforced	79 x 145
1 table cover	150 x 190
4 cellulose towel	19 x 25
2 op-tape	9 x 49
1 adhesive towel	75 x 75
1 extremity drape	230 x 315
1 table cover	150 x 190

Pack Count: 5/10

