

BARRIER® Hand Drape

- 100% impermeable materials providing optimal patient safety
- Optimal folding for aseptic application, easy for one person to use
- Three dimensional design providing improved coverage
- Effective absorption helping to maintain a dry working area



BARRIER® will help you treat more patients to better care

All BARRIER products from Mölnlycke Health Care are designed to offer the optimal combination of protection, functionality and comfort. Our innovative and procedure specific solutions come complete with logistic advantages and expert training and support. These elements together will create a ripple effect of savings in time, effort and costs - helping you treat more patients to better care.

Draping instructions



1
Unfold the drape once and look at the pictogram. The hand on the pictogram illustrates the hand of the patient.



2
Place the unfolded drape over the patient's hand. Unfold the drape towards you. Then unfold the drape away from you, over the patient.



3
Unfold the drape to the sides, over the lower part of the body and then towards the head.

Ref. No.	Description	Disp/Trp
60310	Hand drape 150/370x280 cm, aperture 3,5 cm elastic	7/14
60305	Hand set 1 hand drape 150/370x280 cm, aperture 3,5 cm elastic 1 table cover 150x190 cm	7/14
60306	Hand set 1 mayo stand cover 79x145 cm 1 hand drape 150/370x280 cm, aperture 3,5 cm elastic 1 table cover 150x190 cm	6/12
60307	Hand set 1 table cover 100x150 cmx 1 hand drape 150/370x280 cm, aperture 3.5 cm elastic 1 table cover 150x190 cm 1 wrap	6/12

All specified set components are also available for ProcedurePak®.



Draping instructions



Hold the drape in right position.



Unfold the drape once and look at the pictogram, this hand is the hand of the patient.



Unfold the drape in your direction, so you have a protection between you and the arm/hand table. Then unfold the drape in the other direction.



Unfold the drape to the bottom side and then to the head-side.

Ref. No.	Description	Disp/Trp
60310	Hand drape Hand drape 150/370x280 cm, aperture 3,5 cm elastic	7/14
60305 70310 914080	Hand set 1 hand drape 150/370x280 cm, aperture 3,5 cm elastic 1 table cover 150x190 cm	6/12
60306 996106 70310 914080	Hand set 1 mayo stand cover 79x145 cm 1 hand drape 150/370x280 cm, aperture 3,5 cm elastic 1 table cover 150x190 cm	6/12
60307 914079 70310 914080 -	Hand set 1 table cover 100x150 cmx 1 hand drape 150/370x280 cm, aperture 3.5 cm elastic 1 table cover 150x190 cm 1 wrap	6/12

All specified set components are also available for ProcedurePak®.



XXXXX

