

Hip Drape w/ dislocation bags

- The new design of hip drape with dislocation bags offers an all-in-one draping solution fit for hip surgery and also meets the specific needs of the merging hip arthroscopy procedure
- Dual function integrated bags for dislocation of patient leg and equipped with outlets for liquid disposal and positioned to capture the extensive liquids of arthroscopy surgery
- Integrated strong tube holders for effective attachment of tubes, etc.
- Integrated suction and diathermy pouches on both sides of the drape
- Impermeable all over with extra absorption in critical area

Fit for hip
arthroscopy





1 Unfold the table cover and place it under the patient's leg.



2 Alt. 1: Place an adhesive towel under the leg with the adhesive edge towards the head, remove the release paper. Place a second adhesive towel over the first towel with the adhesive edge away from the patient.



3 Fold one edge around the leg. Fold the second towel over the foot and fix the adhesive edge to the leg.



4 Seal the edges of the package around the foot and along the leg with op-tape.



5 Alt. 2: Cover the leg by using a stockinet, and fix with op-tape.



6 Position the leg through the aperture. Adhesive strip underneath the drape gives extra security.



7 Pull drape towards the head of the patient and secure elastic aperture in correct position.



8 Unfold the drape over the anesthetic frame.

ProcedurePak® Components	
70601	BNS Hip Drape w/dislocation bags 230/300×330cm, ap. 18×22cm elastic, patched
Sterile codes	
60601	Hip Drape w/dislocation bags 230/300×330cm, ap.18×22cm elastic, patched, wrapped
60602	Hip Set w/dislocation bags
996109	1 mayo stand cover reinforced 79×145 cm
–	4 cellulose towel 19×25 cm
914080	1 table cover 150×190 cm
910598	2 adhesive op-towel 75×75 cm
946112	1 stockinet 32×120 cm
911510	2 op-tape 9×49 cm
70601	1 hip drape w/dislocation bags 230/300×330 cm, aperture 18×22 cm elastic
948090	1 table cover reinforced 150×190 cm

