

Hip Set

- Impermeable material
- Easy to drape, all-in-one hip drape featuring elastic aperture
- Strong material
- Integrated transparent leg positioning pouches






01 Start draping with table cover, then drape the leg with stockinet.



02 Position leg through aperture, adhesive strip underneath the drape gives extra security.

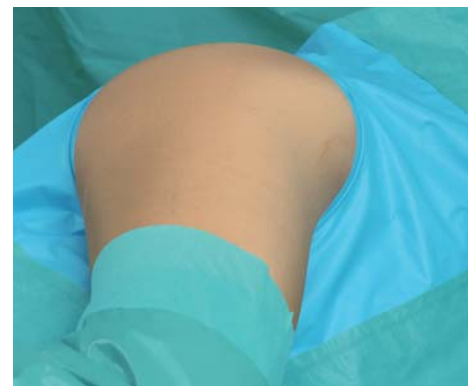

03 Pull drape towards the head of the patient, secure elastic aperture in correct position.



04 Unfold the drape over anaesthesia frame.



05 Integrated in the drape are large leg positioning pouches.

Hip Set / Ref. No. 698285	Size (cm)
4 cellulose towels	19 x 25
2 op-tape	9 x 49
1 adhesive split sheet	150 x 200
1 hip drape, aperture 16 x 19 cm, elastic	230/300 x 350
1 table cover	150 x 190

Pack Count: 5/10

