

Hip Set

- Impermeable material
- Strong material
- Easy to drape, all-in-one hip drape featuring elastic aperture and integrated transparent leg positioning pouches






01 Start draping with table cover, then drape the leg with stockinet.



02 Position leg through aperture, adhesive strip underneath the drape gives extra security.

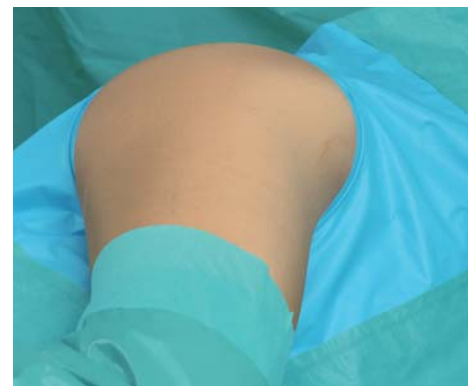

03 Pull drape towards the head of the patient, secure elastic aperture in correct position.



04 Unfold the drape over anaesthesia frame.



05 Integrated in the drape are large leg positioning pouches.

Hip Set / Ref. No. 698286	Size (cm)
1 mayo stand cover reinforced	79 x 145
4 cellulose towels	19 x 25
2 op-tape	9 x 49
1 table cover	150 x 190
1 suction and diathermy bag	38 x 33
1 stockinet large	32 x 120
2 adhesive towel	75 x 75
1 hip drape, aperture 16 x 19 cm, elastic	230/300 x 350
1 table cover	150 x 190

Pack Count: 3/6

