

## Hip Set

- Extra resistant material with strong reinforcement at the operative site
- Improved fluid absorbency around the incision area
- Integrated adhesive edges
- Completely impermeable material
- Integrated adhesive edges
- Accurate folding and packing order to allow aseptic application of the product
- Absorbent comfort layer for the patient





01 Place the adhesive op-sheet or table cover under the patient's leg. Then place a split sheet under the leg with the arrow pointing towards the patient.



02 First unfold the sides of the sheet, and then the lower part. Remove the release paper surrounding the split completely on the side nearest you and to the first crease on the side away from you.



03 Fasten the split under the patient's leg up over the genital area and along the body. Remove the rest of the release paper and fasten the split sheet along the hip and side of the patient.



04 Alt. 1: Place one adhesive towel under the patient with the adhesive towards the patient and remove the release paper. Place a second adhesive towel over the first with the adhesive edge away from the patient.



05 Fold one edge around the patient's leg. Then fold over the second towel and fix the adhesive edge to the patient's leg.



06 Seal the edges of the package around the patient's foot and along the leg with op-tape.



07 Alt. 2: Cover the leg by using a stockinet which is fixed with op-tape.



08 Place an adhesive op-sheet over the patient's body so that the adhesive edge is above the hip area. First unfold the sides of the drape and then over the anaesthetic frame.

Hip Set / Ref. No. 920256	Size (cm)
1 mayo stand cover reinforced	79 x 145
4 cellulose towel	19 x 25
3 op-tape	9 x 49
1 elastic bandage	12 x 6
1 stockinet, large	32 x 120
2 adhesive towel	90 x 75
1 adhesive op-sheet	175 x 175
1 adhesive op-sheet	150 x 240
1 adhesive split sheet, split 20 x 102 cm	200 x 260
1 table cover reinforced	150 x 190

Pack Count: 3/6

