

Knee Arthroscopy Drape

- BARRIER® Knee arthroscopy drape is purpose-designed featuring impermeability all over the drape to minimise the risk of strikethrough in all surgical situations
- Integrated fluid collection pouch for maximized fluid control
- Guided handling points and sophisticated folding principles ensure aseptic application in the draping procedure
- The BARRIER and ProcedurePak® packaging systems ensure correct aseptic handling throughout the entire logistic chain from warehouse to operating theatre





01 Unfold the table cover and place it under the patient's leg.



02 Cover the leg with the adhesive OP-towel and fix it with OP-tape.



03 The leg can also be covered with a stockinet.



04 Place the unfolded arthroscopy drape over the patient's foot. See instruction label for drape direction.



05 Reach through the elastic opening, grasp the patient's foot and pull it through the drape opening.



06 Unfold the lower part of the drape under the patient's leg and place the two elastic openings around the knee.



07 Unfold the upper part of the drape over the anaesthetic frame.



08 Place the leg in correct position for surgery. The draping can also be used with tilted table.

Sterile code		Sterile code	
60101	Knee arthroscopy set	60103	Knee arthroscopy set
70104	1 knee arthroscopy drape 230×320 cm, aperture 6 cm elastic	9214	1 prep-pad 61×112 cm, split 10×36 cm
914080	1 table cover 150×190 cm	996106	1 mayo stand cover 79×145 cm
60102	Knee Arthroscopy set	-	4 cellulose towel 19×25 cm
996106	1 mayo stand cover 79×145 cm	9475	1 split sheet (plastic) 152×193 cm, split 15×61 cm
-	4 cellulose towel 19×25 cm	946111	1 stockinet 22×75 cm
914080	1 table cover 150×190 cm	911510	2 op-tape 9×49 cm
946111	1 stockinet 22×75 cm	70104	1 knee arthroscopy drape 230×320 cm, aperture 6 cm elastic
911510	2 op-tape 9×49 cm	-	1 suture bag 16.5×30.5 cm
70104	1 knee arthroscopy drape 230×320 cm, aperture 6 cm elastic	914080	1 table cover 150×190 cm
914080	1 table cover 150×190 cm		
ProcedurePak® Components		ProcedurePak® Components	
60104	Knee arthroscopy drape 230×320 cm, aperture 6 cm elastic, wrapped	70104	BNS Knee arthroscopy drape 230×320 cm, aperture 6 cm elastic, wrapped

