



BARRIER[®] Knee arthroscopy set

- Integrated fluid collection pouch for effective fluid management
- 100% impermeable materials providing optimal patient safety
- Optimal folding for aseptic application, easy for one person to drape
- Suitable for use also on a tilted table

Draping instruction



1. Start by unfolding the table cover and place it under the patient's leg.



2. Cover the leg with the stockinet. Fold the absorbent comfort layer of the stockinet in on itself to avoid the risk of microbial transfer.



3. Fix it with the OP-tape.



4. Note the direction of the pictogram. Place the unfolded arthroscopy drape over the patient's foot. Reach through the elastic opening, grasp the patient's foot and pull it through the drape.



5. Unfold the lower part of the drape under the patient's leg and place the two elastic openings around the knee.



6. Unfold the upper part of the drape fully towards the head. Use the adhesive patches to secure the film pouch.



7. Place the leg in correct position for surgery. Draping is then completed. The drape can also be used with tilted table.

Scan the QR code for more about Mölnlycke Health Care or to watch a draping instruction video.



Ref. No.	Description	Disp/Trp
60101	Knee arthroscopy set 1 knee arthroscopy drape 230x320 cm, aperture 7 and 5 cm elastic 1 table cover 150x190 cm, 75x190 cm absorbent	6/12
60102	Knee arthroscopy set 1 mayo stand cover 79x145 cm 4 cellulose towels 18x25 cm 1 table cover 150x190 cm, 75x190 cm absorbent 1 stockinet 22x75 cm 2 OP-tape 9x49 cm 1 knee arthroscopy drape 230x320 cm, aperture 7 and 5 cm elastic 1 table cover 150x190 cm, 75x190 cm absorbent	5/10
60103	Knee arthroscopy set 1 preparation pad 61x112 cm, split 10x36 cm 1 mayo stand cover 79x145 cm 4 cellulose towels 18x25 cm 1 split sheet (plastic) 150x193 cm, split 15x61 cm 1 stockinet 22x75 cm 2 OP-tape 9x49 cm 1 knee arthroscopy drape 230x320 cm, ap. 7 and 5 cm elastic 1 suture bag 16x28 cm 1 table cover 150x190 cm, 75x190 cm absorbent	4/8
60104	Knee arthroscopy drape 230x320 cm, aperture 7 and 5 cm elastic	6/12
60105	Knee arthroscopy drape 230/330x320 cm, aperture 7 and 5 cm elastic, wrapped	5/10

All specified set contents are also available for inclusion in ProcedurePak® trays.

Ref. No.	ProcedurePak components
70104	Knee arthroscopy drape 230x320 cm, aperture 7 and 5 cm elastic
70105	Knee arthroscopy drape 230/330x320 cm, aperture 7 and 5 cm elastic, wrapped

Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, SE-402 52 Göteborg, Sweden.
T +46 31 722 30 00. F +46 31 722 34 00. www.molnlycke.com

The Mölnlycke Health Care, BARRIER®, and ProcedurePak® names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies. Copyright (2012)