

Spinal Drape

- Transparent easy-to-use product
- Clear instruction labels for quick draping
- Available in convenient set with table cover and absorbent non-transparent OP-sheet or BNS for your ProcedurePak[®]





1 Hold the drape with the instruction label in the correct direction.



2 Remove the release paper, assisted by one or two persons.



3 Place the drape in the middle of the operation area.



4 Fix the incision foil.



5 Unfold the upper part of the drape over the anesthetic frame and then unfold the drape over the foot end of the op-table.



6 Position the drape and unfold.

	Sterile code
80005471	Spinal set
967775	1 adhesive op-sheet 175×175 cm
80008030	1 spinal drape 240×320 cm, incise area 30×60 cm
948090	1 table cover reinforced 150×190 cm
	ProcedurePak Components
80008030	BNS Spinal Drape 240×320 cm, incise 30×60 cm

