

Vertical Isolation Drape

- Transparent plastic polyethylene film
- For all types of surgery on extension tables
- Different sizes of incise areas
- Integrated instrument pockets, tube holders and fluid collection pouch





1 Hold the drape with the instruction label in the correct direction.



2 Remove the release paper assisted by one or two persons. Note the direction of the arrow.



3 Place the incision foil on the patient.



4 Unfold the drape first up and down, then left and right.



5 Fix the adhesive edge over the frame to the back of the drape.



6 Open the fluid bag and connect suction tube to the outlet if needed.

	Sterile codes
903016	Vertical Isolation Set
996109	1 mayo stand cover reinforced 79×145 cm
-	4 cellulose towel 19×25 cm
815301	1 vertical isolation drape 330×240 cm, incise area 30×40 cm
948090	1 table cover reinforced 150×190 cm
903026	Vertical Isolation Set
996109	1 mayo stand cover reinforced 79×145 cm
-	4 cellulose towel 19×25 cm
815300	1 vertical isolation drape 330×240 cm, incise area 25×86 cm
948090	1 table cover reinforced 150×190 cm
826010	Vertical Isolation Drape 330×240 cm, incise area 25×86 cm
826020	Vertical Isolation Drape 330×240 cm, incise area 30×40 cm
5222	Vertical Isolation Drape 218×320 cm, reinforced 81×40 cm, incise area 35×23 cm

	ProcedurePak® Components
815300	BNS Vertical Isolation Drape 330×240 cm, incise area 25×86 cm
815301	BNS Vertical Isolation Drape 330×240 cm, incise area 30×40 cm
9914	BNS Vertical Isolation Drape 218×320 cm, reinforced 81×40 cm, incise area 35×23 cm